

Figure 1 Sterilized Disposable Blade for pediatric

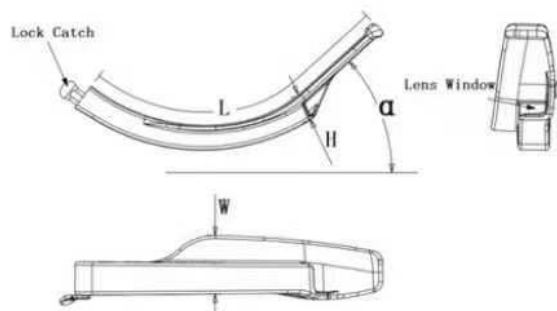


Figure 2 Sterilized Disposable Blade for adult

【Transportation & Storage】

- (1) No heavy pressure, direct sunlight, rain or snow, so as not to damage the Sterilized Disposable Blade and /or Pouch.
- (2) Handle with care .
- (3) Keep away from fire and heat source to avoid deformation of the Sterilized Disposable Blade.
- (4) It should be stored in a ventilated and dry room with non-toxic gas, far away from fire source, with good ventilation, no corrosive gas and relative humidity less than 80%.

【Symbol Directory】

Table2
Symbols used for Sterilized Disposable Blade

	Caution		Batch number
	Manufacturer		Sterilized by Ethylene Oxide
	Do not Re-use		CE mark
	Keep away from sunlight		Fragile,handle with care
	Use By		This way up
	Keep dry		Authorized representative in the European Community
	Do not use if package is damaged		Date for production
	Consult the instructions for use		Sterile Barrier System
	Do not Re-sterilize		Sterile
	Catalog Number		Rx only



Hebei Vimed Medical Device Company, Ltd.
Maikeshan Garment CO., East , North Ring RD.
South, Yanjiao Developing Zone, Sanhe Shi
Langfang Shi, Hebei , 065201 China,
PEOPLE'S RUPUBLIC OF CHINA
VOLKSREPUBLIK
TEL: +86(316)3081859
FAX: +86(316)3081879
Email: export@vimed-mac.com
For additional contact information, please visit
<http://www.vimed-mac.com>



CE0123

User's Instruction

Sterilized Disposable Blade

(Mac Blades)

Basic UDI-DI: 6971834830001AV



Supporting Health Care N. V.
Hoofdkade 90
9561 JK Ter Apel
The Netherlands
TEL: +31 599 700232
Email: Info@supportinghealthcare.eu
<http://supportinghealthcare.eu>

【Product Name】

Sterilized Disposable Blade (also known as Vimac Sterilized Disposable Blade)

【Model/Size】

Pediatric: Mac-00, Mac-0, Mac-1; Adult: Mac-2, Mac-2M, Mac-3, Mac-3D, Mac-3M, Mac-4, Mac-4M.

【Material/Structure】

The Sterilized Disposable Blade is made of medical grade Polycarbonate (PC). It consists of Lock-catch, tunnel, transparent Camera-window, spatula and tip of the blade.

【Intended purpose】

The Sterilized Disposable Blade must be used with the video laryngoscope as its sheath to prevent the video laryngoscope from directly contacting the patient's mouth and to provide better view of patient's airway.

【Indication】

Applied in tracheal intubation of adults and pediatric patients for less than 1 minute in general.

【Contraindications】

There is no absolute contraindication. However, please do not use the sterilized disposable blade if patient is allergic to Polycarbonate (PC). For patients with acute upper respiratory tract inflammation and dyspnea, severe cardiopulmonary disease, hypersensitivity to dicaine, and unexplained severe laryngeal obstruction, please consult an experienced physician before using the video laryngoscope.

【Direction】

- (1) Choose the right model/size of the Sterilized Disposable Blade for the right model/size of the video laryngoscope. And make sure that the Sterilized Disposable Blade is suitable for the patient.
- (2) Open its packing pouch and carefully take out the Sterilized Disposable Blade.
- (3) Put the Sterilized Disposable Blade on the camera rod of the VIMED video laryngoscope, and make sure that the flip-type lock-catch is firmly set on the lock pillar on the video laryngoscope handle.
- (4) After completing the intubation, take off the used Sterilized Disposable Blade from the camera rod of the video laryngoscope, and discard it as medical waste in accordance with local regulations.

【Warning】

- (1) Please read this instruction carefully before using the Sterilized Disposable Blade with video laryngoscopes.
- (2) Inspect the packing pouch carefully before use. If the pouch is broken, discard the sterilized disposable blade immediately.
- (3) Inspect the lock-catch of the Sterilized Disposable Blade before use, if it is damaged, discard the Sterilized Disposable Blade.
- (4) The sterilized disposable blade is single use only. Do not use it repeatedly.
- (5) Discard the used sterilized disposable blade as medical waste in accordance with local regulations.
- (6) Please use the sterilized disposable blade before the sterilization validity (see the seal for the sterilization validity expired date on the pouch).

【Performance】

- (1) Appearance: the surface of the Sterilized Disposable Blade should be clean, smooth, and free from impurities, sharp edges, and burrs; the Sterilized Disposable Blade should be complete and free from damage.
- (2) The dimension of sterilized disposable blade is listed in table 1. The ways of measurement are shown in Fig. 1 and Fig. 2.
- (3) The Sterilized Disposable Blade is sterilized with ethylene oxide and its residue shall not exceed 10 $\mu\text{g/g}$.

Tabelle 1

Model Size	Length L($\pm 3\text{mm}$)	Height H($\pm 2\text{mm}$)	Width W($\pm 2\text{mm}$)	α ($\pm 3^\circ$)	VL Model
Mac-00	72	9	14	36°	PVL-I-01 Plus MC2001 Plus
Mac-0	82	9	16	36°	
Mac-1	95	9	19	36°	
Mac-1M	103	10	19	37°	PVL-I-02 Plus MC2002 Plus VM1202
Mac-2	110	10	19	37°	
Mac-2M	108	15	23	32°	
Mac-3	125	10	23	37°	
Mac-3D	125	11	26	62°	
Mac-3M	125	16	25	37°	
Mac-4	140	14	28	45°	
Mac-4M	138	18	27	40°	